

Community Based Office  
of Otolaryngology Head  
And Neck Surgery

## Reprocessing Flexible Scope



HLD-Nasopharyngoscope  
Infection Prevention

[Nasopharyngoscope.com](http://Nasopharyngoscope.com)  
[info@nasopharyngoscope.com](mailto:info@nasopharyngoscope.com)  
+1-519-502-6161

## Dry Leak Test



## Wet Leak Test



## Dry Storage



## USE

- Rinse
- Scrub with Enzyme sponge
- Rinse
- Fully immerse x 15 minutes
- **Strip test (solution strength)**
- Place in second tube (water) until next use

## FOR RE-USE

- Rinse
- Use
- Repeat the above

## END OF THE DAY

- Store in third tube (dry)

## ALSO

- Change and document (Batch # and Expiry date) HLD solution data as per manufacturer's recommendations.

# HLD-Station Configurations: A modular system adaptable to needs

SHIELD AND GLASS LID ARE OPTIONAL

## Single Scope



## Multiple Scopes



+




## Partial Immersion (HLD of fiber bundle only)



## Add-On Full Immersion



OR

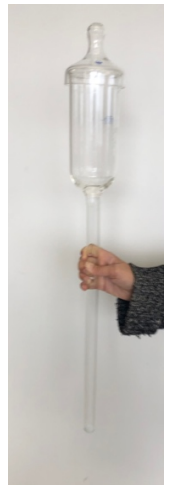

Single unit with wall plate or  
Full Immersion Glass tube